**FLOQSwabs®**  
per autoprelievo vaginale

ITALIANO

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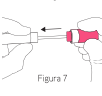
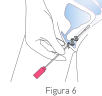
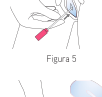
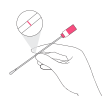
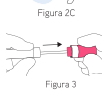
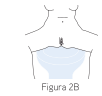
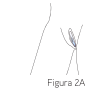
**FLOQSwabs®**  
for self-collection of vaginal swab sample

ENGLISH

**INSTRUCTIONS FOR USE FOR THE PATIENT**

**INTRODUCTION**

FLOQSwabs® is a device consisting of a sterile swab (Figure 1) you can use for self-collection of a vaginal sample. The swab is contained in a container (plastic tube) used to preserve and transport the sample to the laboratory. The swab shaft has a coloured line that indicates the precise point where to place your fingers to facilitate collection. The shaft has a pink cap at one end and a soft tip at the other end for collection.

**NOTE:** Open the swab just before use.

**HOW TO TAKE THE SAMPLE:**

- 1) Before starting self-collection, carefully read the following notes:
  - Do not self-collect a vaginal sample when you are menstruating. Wait at least three days from the last loss of blood.
  - Do not self-collect a vaginal sample during pregnancy or in the three months after giving birth.
  - Do not use vaginal ovules, creams or washes in the three days prior to self-collection.
  - Avoid having sexual intercourse and ultrasound scans or gynaecological examinations in the two days before self-collection.
  - Do not self-collect a vaginal sample if the plastic tube is damaged or open or if the product is past the expiry date shown on label.

- 2) Wash your hands with soap and water, then rinse and dry them. Undress to expose the vaginal area. Put yourself in a comfortable position that allows you to easily insert the swab in the vagina, for example:
  - Standing (Figure 2A)
  - Sitting on the toilet (Figure 2B)
  - Resting one foot on the toilet (Figure 2C)

- 3) Remove the swab from the tube holding it by the pink cap (Figure 3) and place the tube on a clean surface.

- 4) Hold the swab with two fingers at the height of the pink line on the swab shaft. (Figure 4)
 

**NOTE:** Never grip the swab by the pink cap during the collection procedure.

- 5) Using your free hand, spread out the folds of skin (labia) that close the vagina opening. Gently and slowly insert the swab into the vagina until your fingers touch the vulva (Figure 5).
 

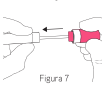
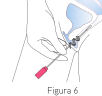
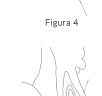
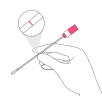
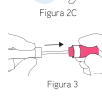
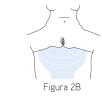
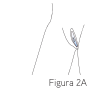
**NOTE:** Do not insert the swab if you encounter resistance. You should be able to insert the swab without any difficulty.

- 6) Gently turn the swab at least 4 times for 10-30 seconds. Make sure that the tip of the swab touches the vagina walls in order to ensure absorption of the sample (Figure 6).

- 7) Extract the swab from the vagina avoiding to touch the external parts of the vagina, insert it in the plastic container gripping it by the pink cap and make sure that you close it securely (Figure 7). Wash your hands with soap and water, then rinse and dry them.

- 8) For sending off the sample, follow the information provided separately.
 

**WARNING:**  
These instructions have been drawn up together with physicians specialized in gynaecology in order to provide all the information necessary to carry out safe self-collection. Should unexpected events or reactions not described in these instructions occur during self-collection, immediately contact your doctor.

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